CHROMATIC STAPH AUREUS

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DESCRIPTION

Chromogenic selective and differential medium for *Staphylococcus aureus* isolation.

METHOD PRINCIPLE

Peptone is a source of amino acids and proteins. The selectivity is due to the concentration of sodium chloride and the composition of the chromogenic mix which allows to differentiate *Staphylococcus aureus* from other staphylococci.

TEST PROCEDURE

Inoculate the surface of the medium by streaking the specimen using a sterile loop. Incubate at 36+/-1°C for 24 hours.

INTERPRETING RESULTS

Staphylococcus aureus grows with pink-mauve colonies; *Staphylococcus xylosus* and *Staphylococcus sciuri* grow with blue turquoise colonies; Gram-negative bacteria are completely inhibited. Final identification must be done by biochemical and serological tests.

QUALITY CONTROL

1. Control of general characteristics, label and print
2. Sterility control
7 days at 25 ± 1°C, aerobically
7 days at 36 ± 1°C, aerobically
3. Microbiological control
Inoculum for productivity: 10-100 UFC/ml
Inoculum for selectivity: 10⁴-10⁵ UFC/ml
Inoculum for specificity: ≤ 10⁴ UFC/ml
Incubation conditions: 24 h at 36 ± 1°C

Microorganisms		Growth
<i>Escherichia coli</i>	ATCC 25922	inhibited
<i>Staphylococcus aureus</i>	ATCC 25923	good

STORAGE

2-8°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product cannot be classified as hazardous under current legislation nor does it contain harmful substances in concentrations ≥1%. It therefore does not require a Safety Data Sheet to be available.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

Liofilchem s.r.l

Via Scozia-Zona industriale - 64026 Roseto degli Abruzzi Tel. +39.085.8930745 - Fax +39.085.8930330
Web site: <http://www.liofilchem.net> E-mail: liofilchem@liofilchem.net

CE IVD
Rev. 0 / 17.04.2008